CLEAR-ALIGNER®

Newsletter 13

Casos clínicos

Case Reports

CLEAR-ALIGNER®

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All cases were carried out by Dr. Pablo Echarri. All the laboratory work was carried out by Ladent laboratory, and the Scheu Dental Technology materials were used.

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© 2014 Centro de Ortodoncia y ATM, Ladent, SL
All rights reserved

This book or any part thereof may not be reproduced, stored in retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying or otherwise, without prior written permission of the publishers.

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Tips and Tricks to Reduce Number of Steps and Treatment Time in CA Clear Aligner Therapy

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Reducing the number of steps in crowding treatments with expansion

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Case 01766: A case presenting the molar Class I malocclusion with crowding in both dental arches. A combined treatment is carried out, with CA Clear Aligner with expansion in maxilla and esthetic first bicuspid-to-first bicuspid brackets and with stripping in mandible.

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
Three CA Clear Aligner steps are necessary for the upper expansion treatment, but here the expansion has been carried out with only one CA Clear Aligner Hard, and Vector 40 Expansion Screw. Each 2-3 days, one turn was carried out, and the necessary expansion was achieved in 5 weeks with only one aligner.

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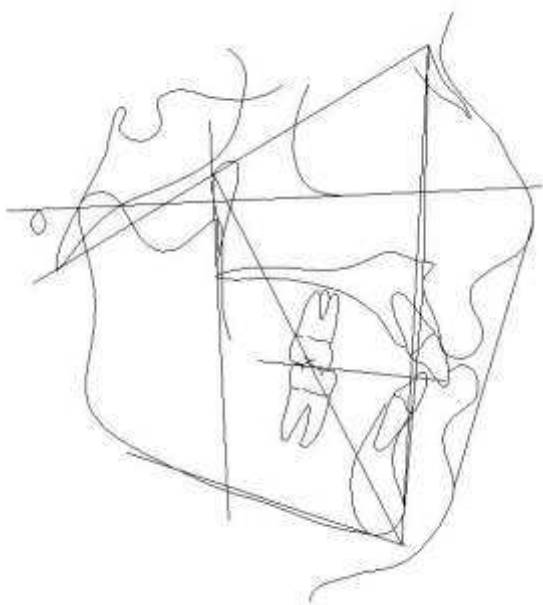Figures 6 and 7: Initial lateral X-ray and initial cephalometric tracing.

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Figures 9 to 12: Insertion of an CA Clear Aligner Hard with Vector 40 Screw for expansion.

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Figures 13 to 17: Two weeks of treatment. Upper expansion control and bonding of the lower brackets.

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Figures 18 to 21: Five weeks of treatment with Expansion CA Clear Aligner.

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Figures 22 to 26: Five weeks of treatment. Observe the expansion achieved with only one aligner and in a short treatment time.

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Figures 27 to 31: Continuation of the treatment with CA Clear Aligner to close the spaces and to align the teeth.

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Reducing the number of steps in crowding treatments with stripping

Case 01701: Case presenting a molar Class I malocclusion with crowding in mandible, and anterior open bite. The treatment was carried out with CA Clear Aligner in mandible, and lower canine-to-canine stripping, to correct crowding, and then the the lower incisors were retroclined to close the anterior bite. The Progressive Stripping Technique by Dr. Echarri (see Newsletter Nº12), indicates the expansion to create space, facilitating the stripping procedure.

Two CA Clear Aligner steps may be necessary to create space, but also, only one aligner can be used and activated with CA-Tip 1 Plier for space opening, or using the elastomeric separators, inserted at the office half an hour before the stripping.

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Mosquito tweezers are used to place the separators.

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Figuras 32-36: Initial intraoral photographs.

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Figure 37: Initial panoramic X-ray.

Figuras 38-40: CA Clear Aligner de expansión oblicua lateral.

Figures 38-40: CA Clear Aligner for oblique-lateral expansion.

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
Figures 41-43: CA Clear Aligner activation using CA-Tip 1 Plier, to create space. The activation is carried out in interdental space, on labial and on lingual side.

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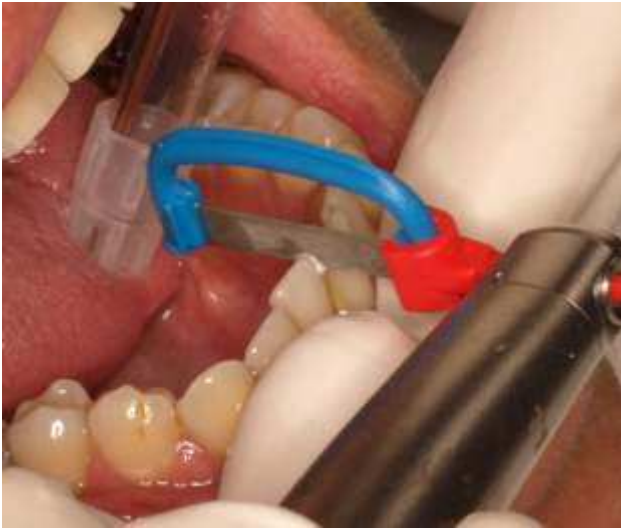Figures 44-47: Insertion of an elastomeric separator with Mosquito tweezers.

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