CLEAR-ALIGNER®

Newsletter 14

CLEAR-ALIGNER®

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All cases were carried out by Dr. Pablo Echarri. All the laboratory work was carried out by Ladent laboratory, and the Scheu Dental Technology materials were used.

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© 2014 Centro de Ortodoncia y ATM, Ladent, SL
All rights reserved

This book or any part thereof may not be reproduced, stored in retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying or otherwise, without prior written permission of the publishers.

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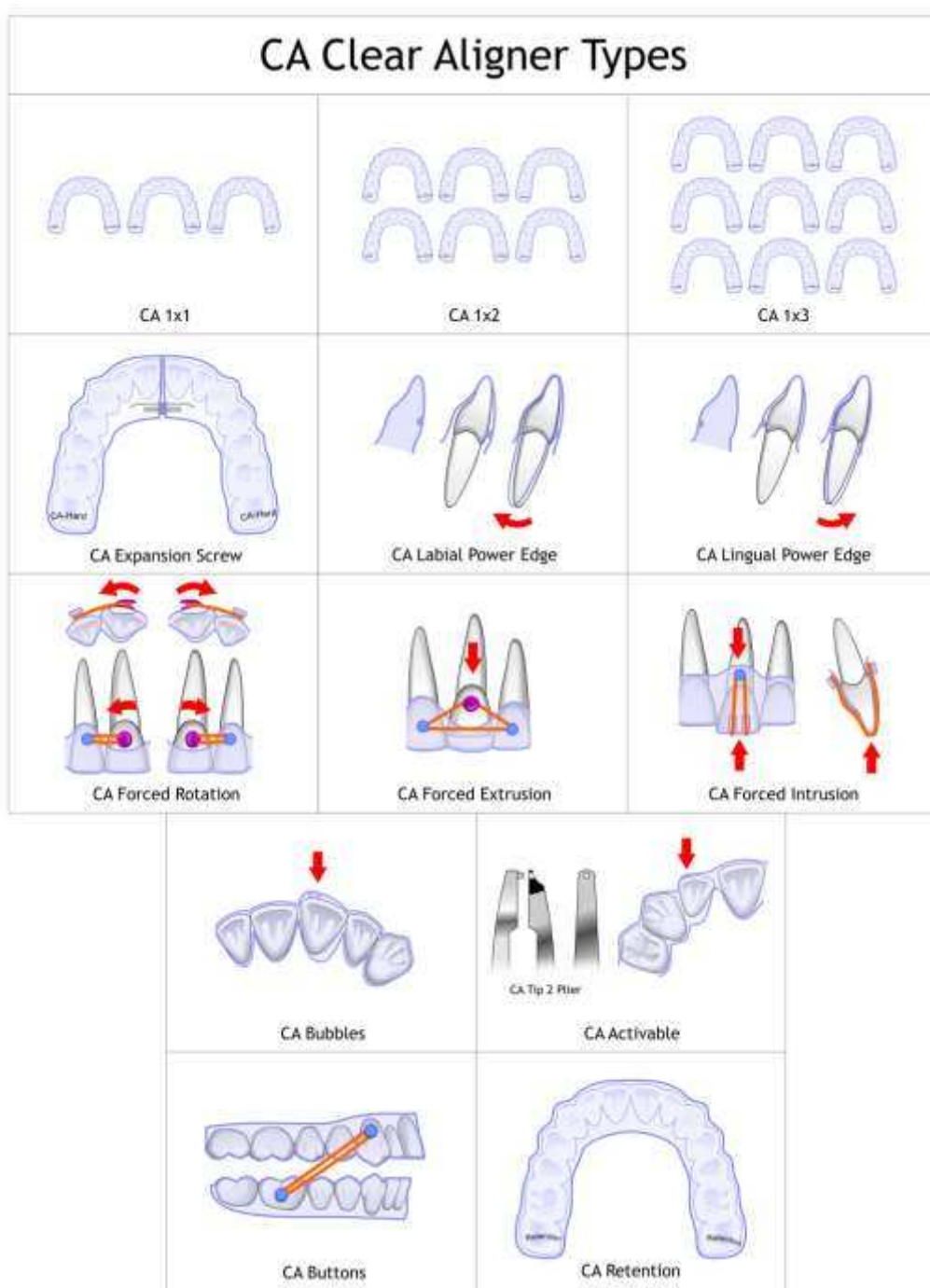
In this Newslewtter, we are going to talk about the new types of CA Clear Aligner, the new CA Power Grips, and the new CA Tip Pliers.

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In 1x1 protocol, one step is carried out on the basis of one impression, in 1x2 protocol, two steps are carried out on the basis of one impression, and 1x3 protocol consists of 3 steps carried out on the basis of one impression. The protocols 1x2 and 1x3 are recommended in expansion or space closure stages, while they are not recommendable in finishing stage.

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The CA Vector40 Screw is recommended in expansion treatment, because it reduces the number of steps (see Newsletter 13).

The CA Labial Power Edge is used to increase the torque, and it can be ordered from the laboratory, or it can be carried out in the clinic, using the CA Tip Plier 4 (see details in the further text of this Newsletter). The CA Lingual Power Edge is used to reduce the root torque, and it can also be ordered from the laboratory or carried out with the same plier.

The CA Forced Rotation and CA Forced Extrusion are carried out with CA Power Grips 1 and 1/8" and 4.5 oz elastics (see details in the further text).

The CA Forced Intrusion is ordered from the laboratory. It has labial and lingual buttons, and the grooves for the elastic to apply the force to the tooth (1/8" and 4.5 oz).

The CA Bubbles allow more effective movements. CA Soft, CA Medium, CA Hard and CA Bubbles are ordered from the laboratory. CA Bubbles has a space which allows the movement of the tooth, and the activation to achieve this movement.

The CA Activable consists of CA Soft and CA Medium, carried out with the set-up. CA Hard has a space created to allow the movement, but it is activated in the clinic with CA Tip Pliers (see details in the further text).

The CA Buttons are ordered from the laboratory, or the buttons can be bonded on the CA Hard in the clinic and intermaxillary elastics can be used to finish the correction of overjet or midline.

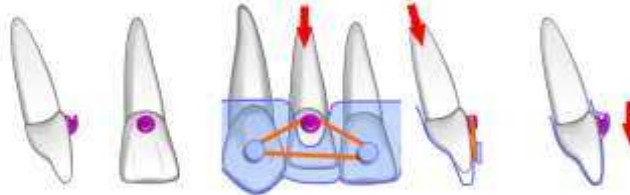
The CA Retention is used as a retainer, and it is more resistant.


CLEAR-ALIGNER®

CA Power Grips


Power Grip #1
MFM CA Power Grip - Multifunctional Forced Movement - Extrusion

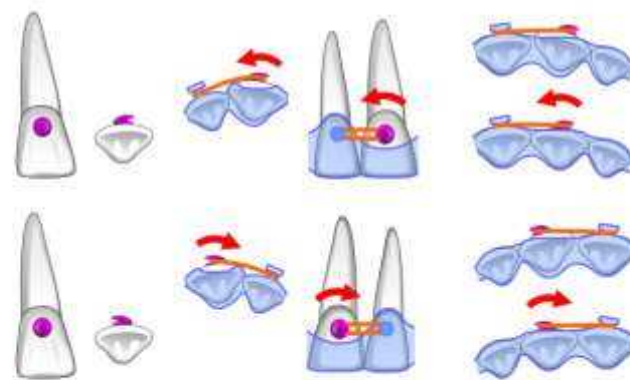
Power Grip #1
MFM CA Power Grip - Multifunctional Forced Movement - Rotation

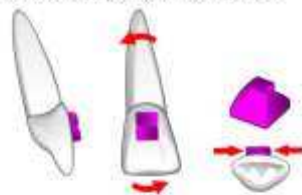
Power Grip #2
MDI CA Power Grip - Mesial/Distal Inclination

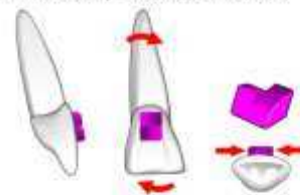
Power Grip #3
DMI CA Power Grip - Distal/Mesial Inclination

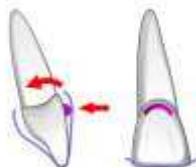Power Grip #4
FALA CA Power Grip - Force Application Labial (Standard)

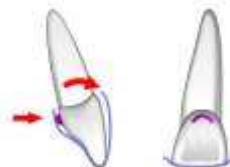Power Grip #5
FALI CA Power Grip - Force Application Lingual (Small)

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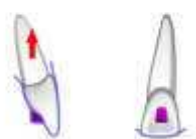
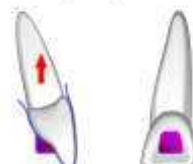
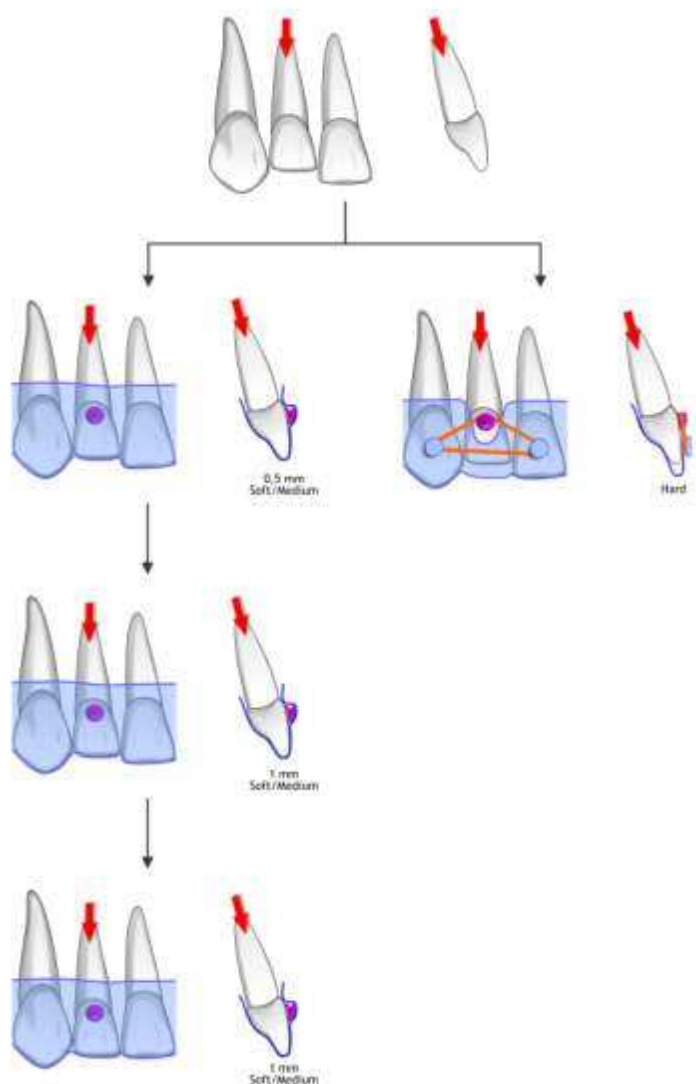
El CA Power Grip 1 Multifuncional Forced Movement (MFM) is used for forced extrusion and rotation.

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
CA Power Grip 1 can be used for forced extrusion, and as it can be seen in this scheme, a patient should use CA Clear Aligner with elastics during the night and at home, and when he goes out, he should use CA Clear Aligner Soft and Medium which cover the CA Power Grip and offer more esthetics. In this way, a more rapid movement is achieved.

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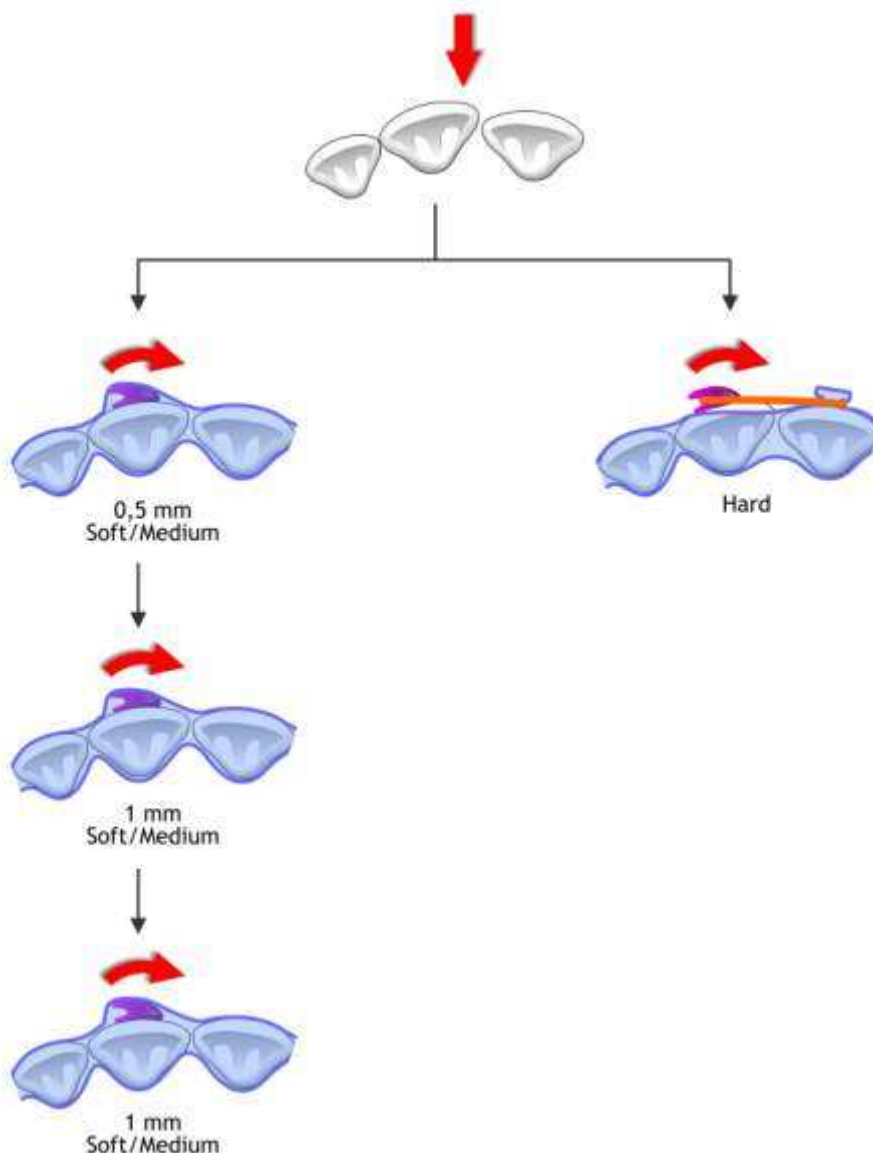
CA Power Grip 1 is also used for forced rotation. Also, CA Clear Aligner with elastics is used during the night and at home, and when a patient goes out, he should use the CA Clear Aligner which covers the CA Power Grips.

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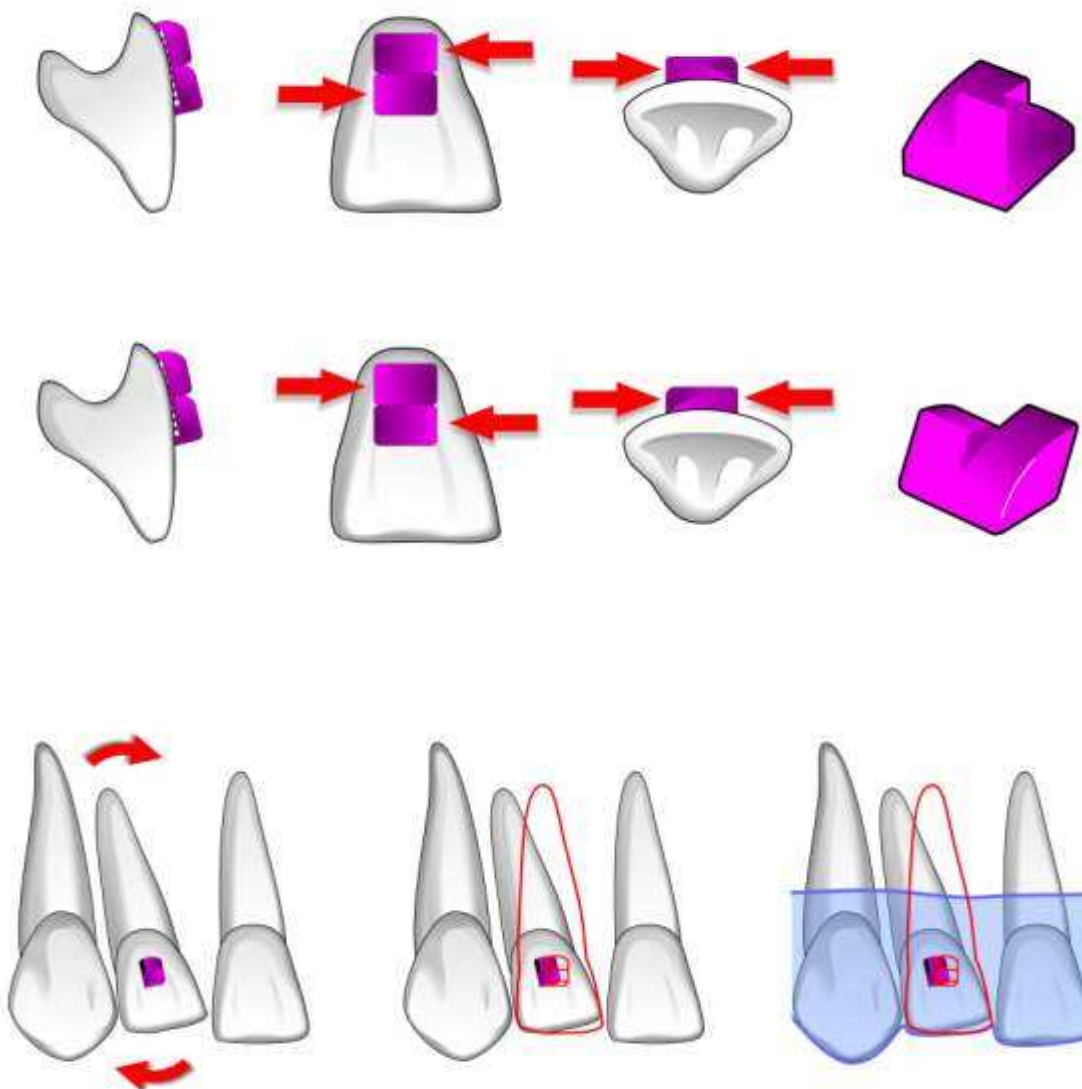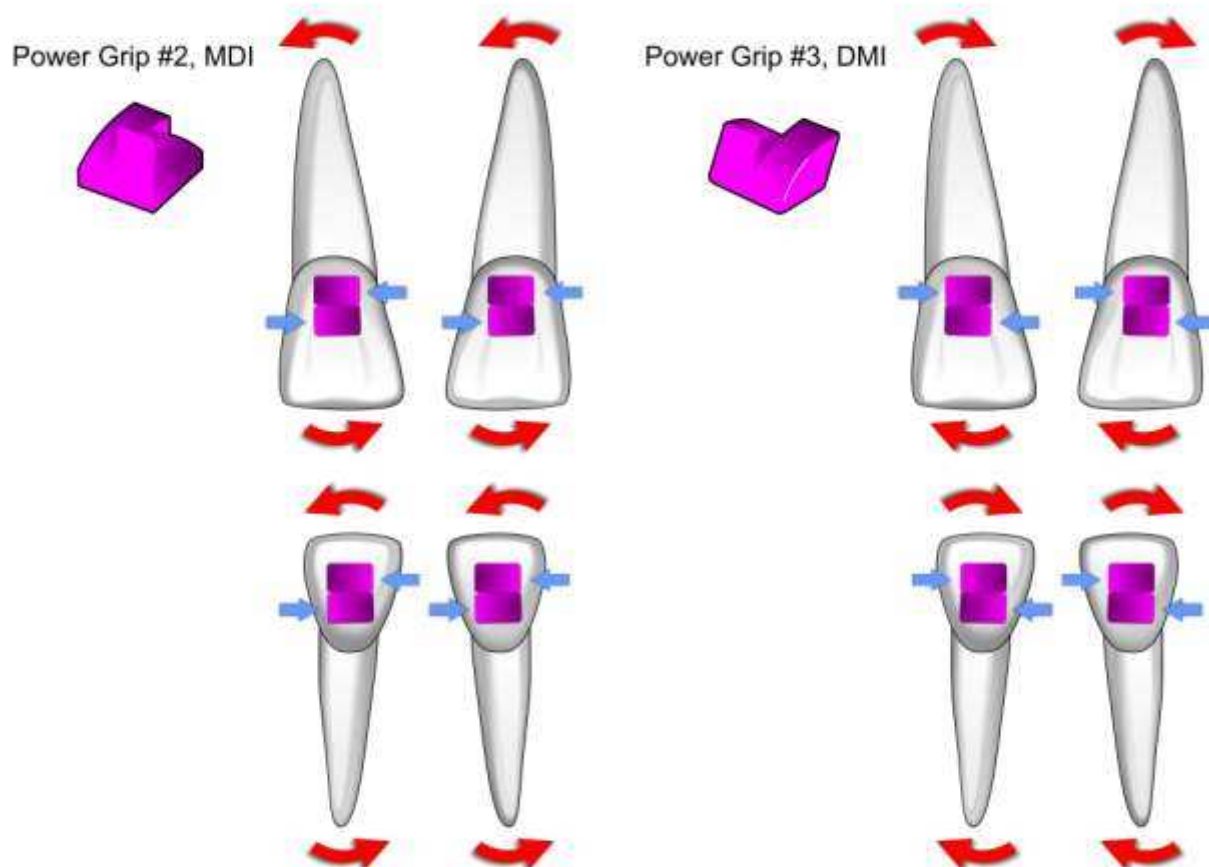
The CA Power Grips 2 and 3 allow the correction of mesio-distal inclinations. They should be bonded to the tooth before taking the impression which is sent to the laboratory.

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
CA Power Grip 2, Mesio-Distal Inclination (MDI), is used to carry out a root movement to distal in upper right and lower left teeth. In upper left and in lower right teeth, it carries out root movement to mesial.

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
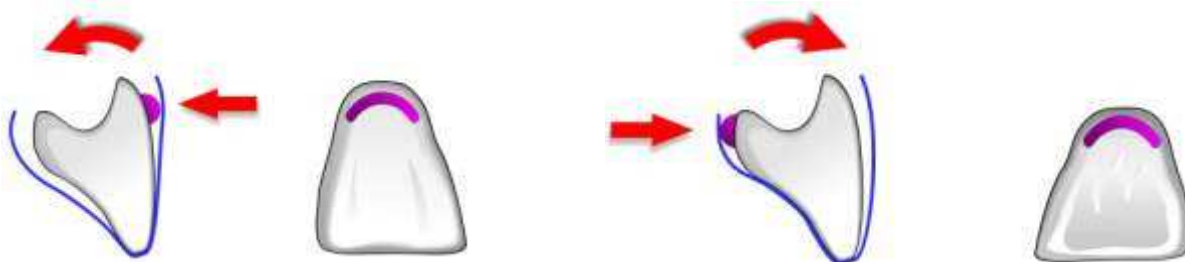
CA Power Grip 2, Disto-Mesial Inclination (DMI), is used to carry out a root movement to mesial in upper right and lower left teeth. In upper left and in lower right teeth, it carries out root movement to distal.

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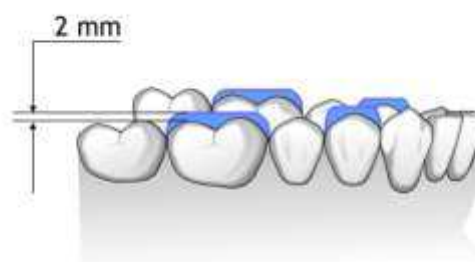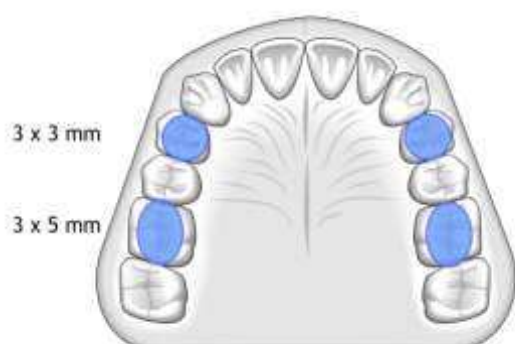
The CA Power Grips 4 and 5 allows the correction of root torque. They should also be bonded to the tooth before taking the impression which is sent to the laboratory.

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
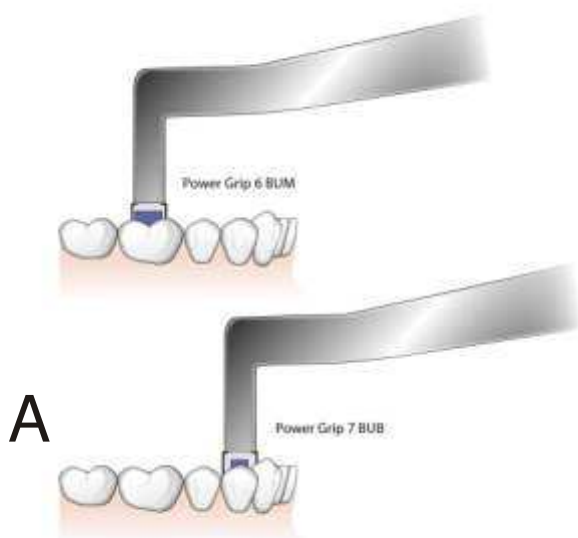
The CA Power Grips 6 and 7 allow us to carry out posterior build-ups to increase vertical dimension.

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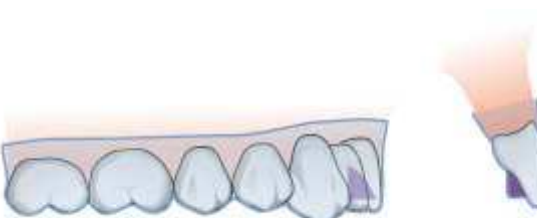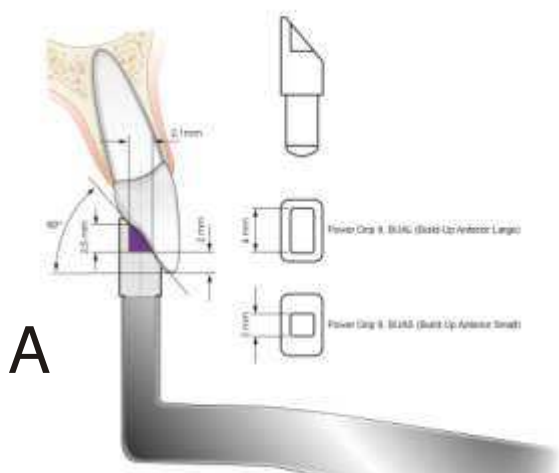
They are carried out easily before taking the impression, as it can be seen in the figure A. The occlusion should be checked with articulating paper. The figure B shows a scheme of CA Clear Aligner carried out over the build-ups in posterior teeth.

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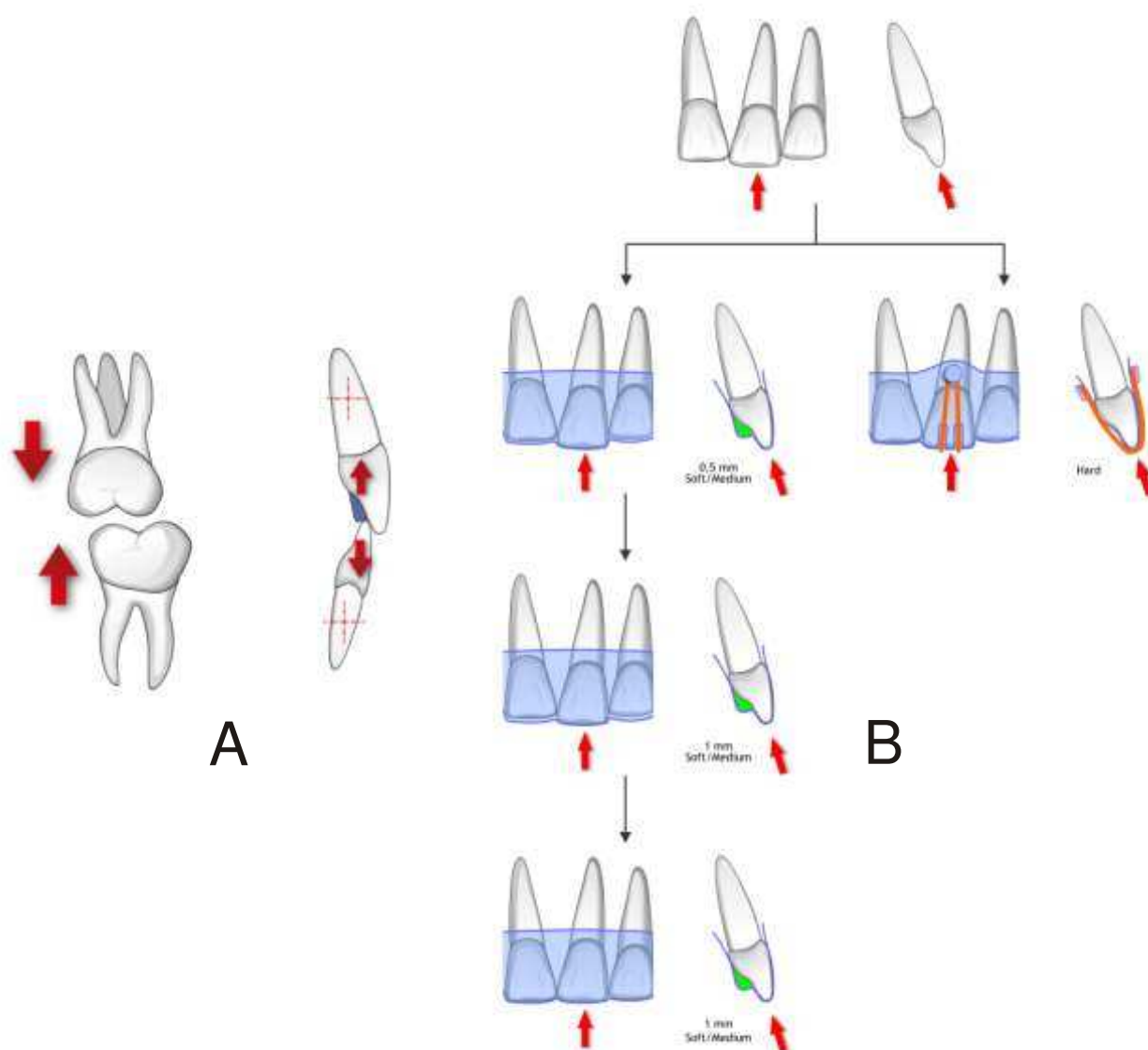The new CA Power Grips 8 and 9 allow us to carry out anterior build-ups in cases of anterior deep bite. They should be bonded in four upper incisors, at the distance of 2 mm from the incisal edge, as it can be seen in the figure A. The figure B shows a scheme of CA Clear Aligner adapted over the anterior build-ups.

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A: Anterior build-ups facilitate the correction of deep bites because the intrusion force of incisors passes closer to the resistance center of these teeth, and posterior teeth disocclusion facilitates its extrusion.

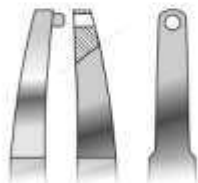
B: Anterior build-ups can also be combined with CA Forced Intrusion. A patient should use CA Clear Aligner and elastics during the night and at home, and when he goes out, CA Clear Aligners for intrusion without elastics.

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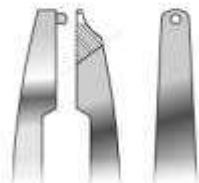
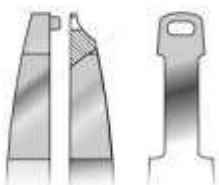
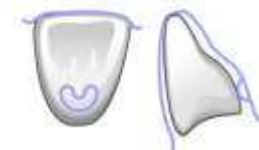
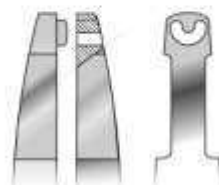
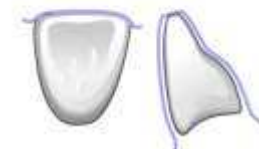
The CA Tip Pliers do not need to be heated to mold CA Clear Aligners, and therefore, no torch nor thermometer.

The CA Tip Plier 1 is used to increase the CA Clear Aligner retention (if necessary), applying more pressure in the space between the contact point and interdental papilla.

The CA Tip Plier 2 is used basically for rotation movement activation, incorporating the bubbles in mesial or distal of the teeth, both in labial and lingual. They will have an effect only if the laboratory is asked to create a space for this movement.

The CA Tip Pliers 3 and 4 are new ones. The CA Tip 3 Plier is used for labio-lingual crown inclinations. The activation is carried out near to the incisal edge on labial, or on lingual side. It is also necessary that the laboratory creates a space for this movement. The CA Tip 4 is used to activate root torque, and the activation is carried out in gingival zone on labial or on lingual side. The laboratory also should create a space for this movement. They have the same function as CA Power Edges or CA Power Grips 4 and 5.