CLEAR-ALIGNER®

Newsletter 15

CLEAR-ALIGNER®

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All cases were carried out by Dr. Pablo Echarri. All the laboratory work was carried out by Ladent laboratory, and the Scheu Dental Technology materials were used.

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© 2014 Centro de Ortodoncia y ATM, Ladent, SL
All rights reserved

This book or any part thereof may not be reproduced, stored in retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying or otherwise, without prior written permission of the publishers.

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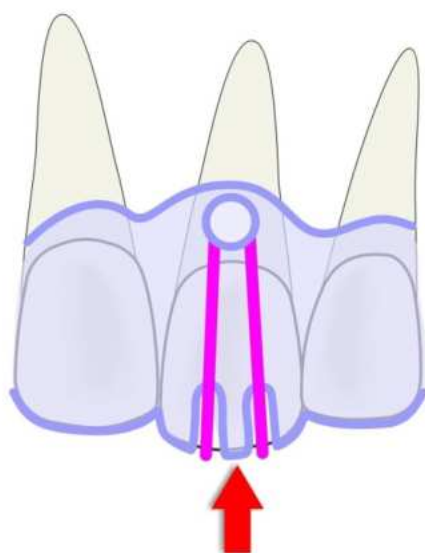
Figura 1. Esquema del CA Forced Intrusion

CA Clear Aligner Forced Intrusion

CA Clear Aligner can be used in forced intrusion of a single tooth without progressive set-ups, which saves both time and steps in a treatment.

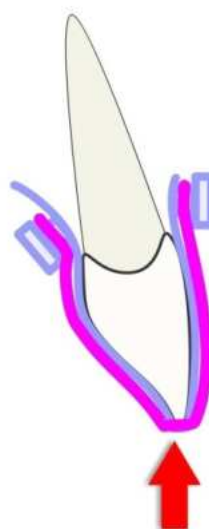
Figure 1. CA Forced Intrusion Scheme.

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CA Forced Intrusion consists of bonding of CA buttons on the CA Clear Aligner Hard (0.75 mm/.030") molded over the original model without carrying out the set-up models.

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CA Buttons should be bonded both on lingual and labial side of the tooth to intrude and:

1. In anterior teeth, the grooves are carried out so the elastic can apply the force on the tooth to intrude it.
2. In posterior teeth, the posterior part of the aligner is trimmed.

Before carrying out intrusion of the tooth, you should check:

1. If there is enough space between the neighbouring teeth for intrusion. If not, the space should be gained before carrying out the vertical movement. The space can be created with expansion (progressive set-up models) or with stripping.
2. If the torque of the tooth to intrude is correct, to avoid that the apex of tooth contacts the labial and lingual cortical bone, and that the intrusion can be carried out through the spongy bone.

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
Figure 2. Torque control during the intrusion.

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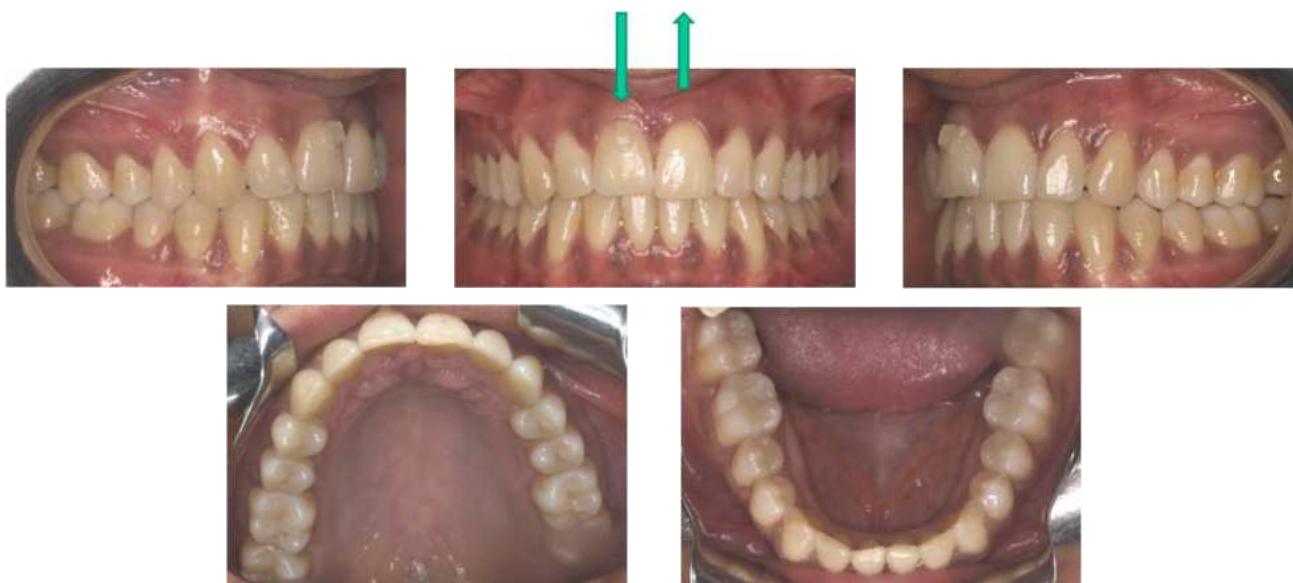CASE 1: Female adult patient with an indication of upper right central incisor extrusion, and upper left central incisor intrusion to level the incisal edges and gingival margins (Figs. 3 to 8).

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Figures 3 to 7. Initial photographs of Case 1 in which upper right central incisor extrusion and the upper left central incisor intrusion should be carried out.

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A button is bonded on gingival side of upper right central incisor labial surface. CA Clear Aligner Forced extrusion is inserted for the right incisor, and CA Clear Aligner Forced intrusion for the left incisor (Figures 9 to 12).

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Figures 9 to 12. The CA Clear Aligner Forced extrusion for the right incisor, and the CA Clear Aligner Forced intrusion for the left incisor.

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Figures 13 to 16. A CA Clear Aligner Forced extrusion scheme for the right incisor can be seen. CA Clear Aligner Hard is carried out (a complete extrusion of the tooth has been carried out on the set-up model to make space and to extrusion guide), and CA Power Grip 1 (MFM - Multifunctional Forced Movement) is bonded in the gingival part of the labial side of the incisor. Two CA buttons are bonded on the CA Clear Aligner and the use of 2.5 oz or 4.5 oz (1/8" or 3/16") elastics is indicated.

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Figures 17 to 20. A CA Clear Aligner Forced intrusion scheme can be seen. It can be seen that it is not necessary to carry out a set-up model with the left incisor movement, but the buttons are bonded in labial and in palatal of CA Clear Aligner Hard. Also, two grooves are carried out in the incisal edge so the elastic can apply the force on the tooth. The use of 2.5 oz and 1/8" elastics is indicated for tooth intrusion, as it can be seen in the image. The elastics should be changed at least once a day.

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The figures 21 to 23 show the treatment result after 8 weeks.

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CASE 2: Female adult patient who presents the absence of the third molars and the lower right second molar. As a consequence, an upper right second molar extrusion took place. The prosthetic repositioning over the implant of the lower right second molar is indicated, but the intrusion of the upper right second molar must be carried out in order to provide space for the crown (Figures 24-27).

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Figures 28 to 31. The CA Clear Aligner Hard carried out over the initial model (without set-up model). The buttons are bonded on labial and palatal side of the molar. Also, the occlusal part of the aligner is trimmed on the occlusal surface of the molar to be intruded. The use of 1/8" and 2.5 oz elastics is indicated during the first 2-3 weeks, and then 4.5 oz. The molar is intruded 1.5 mm in 8 to 10 weeks. The elastics should be changed each day.

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Figures 32 to 37. CA Clear Aligner Forced molar intrusion in the mouth.

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Figures 38 to 41. Final photographs after the molar intrusion.

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Conclusions: CA Clear Aligner Forced intrusion allows us to carry out effectively the intrusion movements with a reduced treatment time and reduced number of steps.