CLEAR-ALIGNER®

Newsletter 8

Casos clínicos

Case Reports

CLEAR-ALIGNER®

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All cases were carried out by Dr. Pablo Echarri. All the laboratory work was carried out by Ladent laboratory, and the Scheu Dental Technology materials were used.

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© 2013 Centro de Ortodoncia y ATM, Ladent, SL
All rights reserved

This book or any part thereof may not be reproduced, stored in retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying or otherwise, without prior written permission of the publishers.

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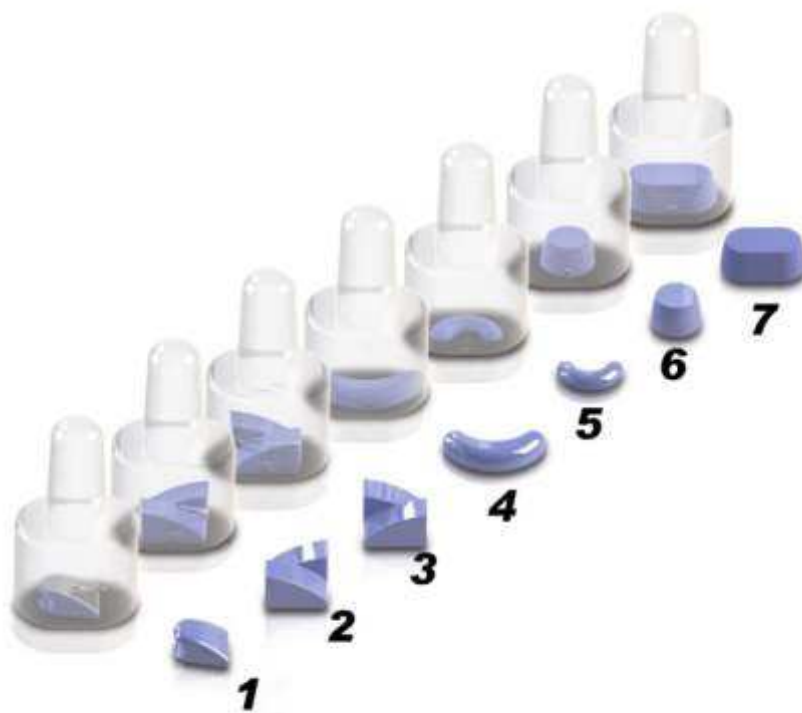
1. MFM Multifunctional Forced Movements
2. MDI Mesial/Distal Inclination
3. DMI Distal/Mesial Inclination
4. FALA Force Application Labial
5. FALI Force Application Lingual
6. BUB Build-Up Bicuspид
7. BUM Build-Up Molar

The CA Power Grips were designed by Dr. Pablo Echarri, developed by Scheu-Dental, and tested by Ladent, SL and Echarri Clinic. The CA Power Grips consist of seven molds which allow easy attachment of composite retentions to the teeth, which facilitates certain dental movements with the CA Clear Aligner technique. The seven CA Power Grips are:


1. MFM Multifunctional Forced Movements
2. MDI Mesial/Distal Inclination
3. DMI Distal/Mesial Inclination
4. FALA Force Application Labial
5. FALI Force Application Lingual
6. BUB Build-Up Bicuspид
7. BUM Build-Up Molar

CA Power Grip 4: Force Application Labial (FALA) CA Power Grip 5: Force Application Lingual (FALI)

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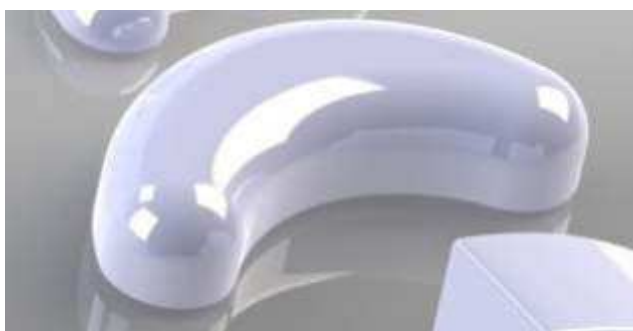In this Newsletter, we are going to talk about the CA Power Grip 4 (Force Application Labial - FALA) and CA Power Grip 5 (Force Application Lingual - FALI), used to correct the torque in approximately 5°.

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CA Power Grip 4 is bigger and it can be used in labial surface of upper incisors and upper and lower canines to increase the torque of these teeth.

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CA Power Grip 5 can be bonded in labial surface of upper lateral incisors and in labial surface of the four lower incisors, to increase the torque of these teeth. It can also be bonded in palatine surface from upper/lower canine to upper/lower canine, to reduce the torque of these teeth.

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CASE 01095

Initial photographs of the case with the negative torque of lower central right incisor. The root of this tooth provokes the swelling of alveolar edge at this level.

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The impressions are taken and the torque of this tooth is increased in the set-up model. The torque is increased for 5°, controlled by CA Checker. Also, the lingual surface of this tooth is blocked with a Blokkout Resin layer, progressively thicker from incisal to gingival. If the impression is taken after the bonding of Power Grip, the model should be trimmed so CA Clear Aligner can carry out the pressure at this level.

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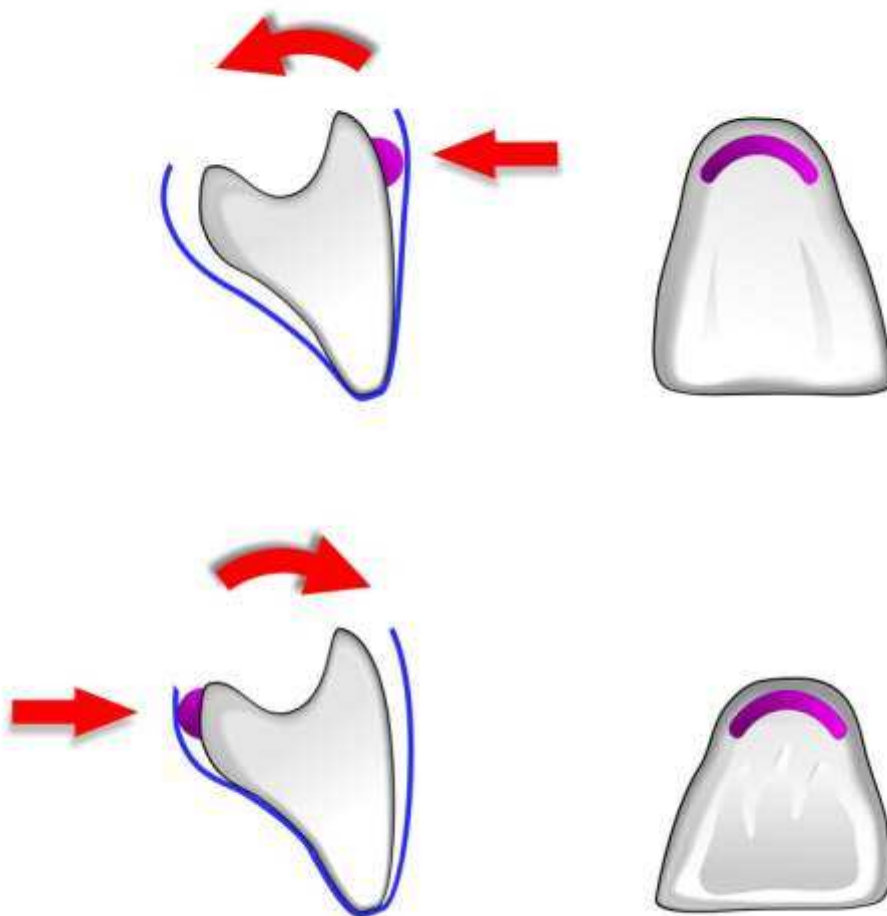
This scheme shows how the Power Grip works, and how the pressure is carried out at the level of the Power Grip. The movement is possible because the space has been created when the CA Clear Aligner was made.

Procedimiento de cementado de CA Power Grips 4 y 5 *Bonding procedure of CA Power Grips 4 and 5*

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*Isolate the working field with cotton rolls
separators and aspire.*

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*Etch with orthophosphoric acid during 20-
30 seconds.*

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** We recommend: Light Bond from Smile Dental, Tetric Eco Ceram from Ivoclar Vivadent, and Tetric Evo Flow from Ivoclar Vivadent.*

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*CA Power Grip 5 bonded on the tooth.
Frontal view.*

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*CA Power Grip 5 bonded on the tooth.
Lateral view.*

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*CA Power Grip 5 bonded on the tooth.
Occlusal view.*

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The alignment treatment in both arches using CA Clear Aligner has also been carried out. In these photographs, the final result can be seen. The upper midline matches with the center of a lower incisor, because the patient had only 3 incisors. The crown of upper right canine has been professionally replaced.

Antes/ *Before*

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Antes/ *Before*

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CLEAR-ALIGNER®